HYA*corp*

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BioScience GmbH
Rheinstr. 96 · 56235 Ransbach-Baumbach
Germany

Stand der Information: 12. 2008

For Restoration of Volume and Contouring of Body Surfaces. High Viscosity.

Instruction for Use (IFU)

Description

HYAcorp is a clear and viscous gel supplied in a special syringe of 10 ml. The product is for single use only. The gel consist in a form of a cross-linked hyaluronic acid.

Composition

1 ml of **HYAcorp** contains
 Na-hyaluronate, cross-linked (Hylan Gel) 20 mg
 Sodium chloride 6.9 mg
 Water for injection ad 1 ml

Indication and usage

HYAcorp is intended to be used for volume restoration and contouring of body surfaces. The depth of the injection may vary from subcutaneous to supraperiosteal administration depending on the treatment site.

- Buttocks
- Calves
- Correction of concave deformities
- Regio zygomatico

Body shaping with **HYAcorp** means that the bottom and other body parts can be filled out in a natural manner. **HYAcorp** is used the shaping of the décolletage, amongst other things, but also as a supplement after plastic surgery procedures such as liposuction. The gel is injected deep into the skin and lifts the tissue in a natural way.

The injection should be made by physicians with the knowledge and the experience of fat transplantation and similar treatments.

The duration of the filling effect can vary and is depending of the depth and area of injection.

Warning

HYAcorp is only intended for subcutaneous to supra-periosteal administration. Do not inject intravascularly.

Precautions

Normal precautions associated with subcutaneous and/or supraperiosteal injections must be observed. Like any such procedure, implantation of **HYAcorp** is associated with an inherent risk of infection.

A thorough anatomical knowledge of the treatment site is mandatory and special caution should be exercised when treating areas in close proximity to vulnerable structures, e.g. nerves, vessels and viscera.

Patients who are using substances that affect coagulation, such as aspirin and non-steroid anti-inflammatory drugs may, as with any injection, experience increased bruising or bleeding at injection site.

HYAcorp should not be used in or near anatomic sites where there is active skin diseases, inflammation or related conditions.

HYAcorp should not be injected into a area where a permanent implant has been placed.

HYAcorp have not been tested in pregnant or lactating women or children.

The treated area should not expose to intense heat or extreme cold at least until the initial swelling and redness have solved.

Patients should advised to avoid strenuous activity, including constant pressure in the treated area for a few weeks following the treatment.

Side effects

After the injection of **HYAcorp**, some common injection-related reactions may occur. These reactions include erythema, swelling, tenderness, pain, bruising or itching on the implant site. Typically resolution is spontaneous within one to two weeks.

Adverse events

Uncommon, delayed inflammatory reactions and implant complications such as swelling and indurations have been reported. Erythema and tenderness may also occur.

In pronounced cases a short course of oral corticosteroids may prove effective.

Cannulas

For safe use of **HYAcorp** it is important to use an appropriate cannula like 16 G blunt or 16 G angio catch with LuerLock.

Dosage and administration

The injection technique with regards to the depth of injection and the administered quantity may vary in the different cases and needs for volume augmentation.

The injection should be made by physicians with the knowledge and the experience of fat transplantation and similar treatments.

Only correct to 100 % of the desired volume effect.

Do not overcorrect.

Note: The syringe and any unused material must be discarded directly after the treatment session.

How supplied

HYAcorp is supplied in a 10 ml syringe with LuerLock and blistered in a Medipeel-Pouch. A patient record label is a part of the Medipeel-Pouch label. The content of the syringe and the outside of the syringe are sterile. For that reason, the confection of **HYAcorp** can be used in the operating room.

Shelf life and storage

The shelf life of **HYAcorp** is 24 months. The storage temperature can be between 2 °C – 25 °C.

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